Smartee®

www.smarteealigners.com

Patient Guide-Smartee SA⁺

P A T I E N T G U I D E

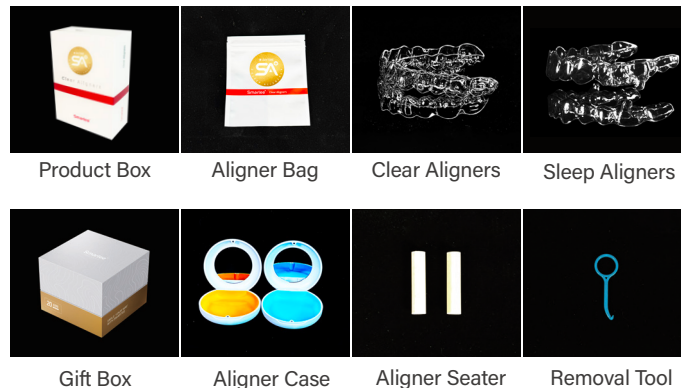
Smartee **SA** PLUS

Contents

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Introduction

Thank you for choosing the Smartee SA PLUS product series. Congratulations on embarking on a wonderful journey towards effortless nighttime breathing and a confident daytime smile! This guide will walk you through using your Smartee Sleep Aligners and Smartee Clear Aligners, ensuring a comfortable and pleasant treatment process along with effective snoring control. Upon receiving your aligners, the package typically includes the following items:

Start Wearing Aligners

The Smartee SA PLUS product series includes both nighttime Sleep Aligners and daytime Clear Aligners, designed to address the dual needs of managing snoring during sleep and facilitating orthodontic correction during waking hours.

The initial shipment will include five sets of nighttime Sleep Aligners based on the titration protocol, which is designed to identify your optimal mandibular advancement position. Once this position has been confirmed, you will receive the complete set of both nighttime Sleep Aligners and daytime Clear Aligners.

① Check the label

Verify the name on the packaging label to confirm that the product box you have received is intended for you.

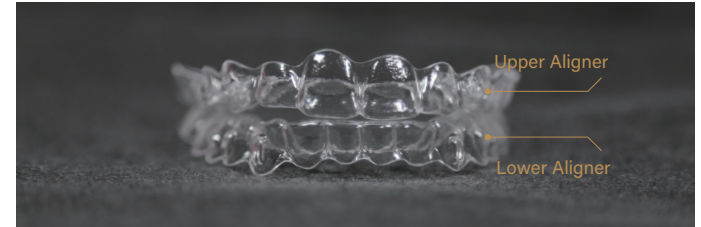
"Daytime/Nighttime" indicators on the label distinguish between Clear Aligners for daytime use and Sleep Aligners for nighttime wear.

② Know your aligners

Note the difference between the upper and lower aligners:

- The upper aligner is bigger than the lower aligner.
- There are two obvious bulges designed to fit your front teeth.


③ Wear your aligners from Stage 1

- You will receive some of your aligners or all of your aligners, depending on your doctor's preference.
- The wearing sequence varies at different stages of treatment. Please refer to the instructions in the "Wear Time" section on page 07.

Wearing & Removal of Clear Aligners

Wearing

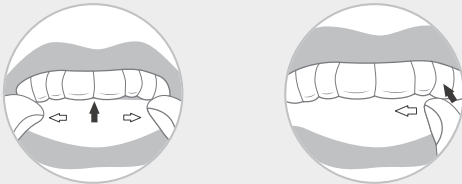- Clean your mouth and hands thoroughly before wearing.
- Place your aligners in position following a "from middle to sides" sequence with your fingers.
- Start by placing the aligners over your front teeth and push the aligner down gently.
- Press the aligner gently over your molars with your fingers until it fits to your teeth.

Note :

Remove the aligners before eating or consuming any beverage other than cold water. Sleep Aligners are intended for use exclusively during sleep.

Tips

- It is common to experience mild discomfort or notice slurred speech during the initial few days. Don't worry – these symptoms are usually short-lived and should resolve gradually in the next couple of weeks.
- If you experience severe pain or unusual discomfort, contact your doctor immediately.
- We recommend avoiding drinking colored or carbonated drinks such as coffee, wine, tea, soft drinks, to prevent tooth decay and aligner staining.
- We recommend reducing smoking throughout the treatment.

Removal

- Use your fingertips to gently pull the aligner off one side of your back teeth, then pull off the other side of the back teeth to loosen the aligner.
- Continue to remove the aligner towards the front teeth and lift it off your front teeth.
- A removal tool can also be used. Use the small hook of the removal tool to separate the aligner from one side of the back teeth, and gently apply force to loosen the aligner.

Tips

- Be gentle! To maintain the integrity of your aligner, avoid applying excessive pressure, pulling, or twisting.
- After removal, rinse them briefly, shake off any excess water, and store them in the aligner case.

Wearing Time

Follow your doctor's instructions.

The recommended wear schedule is as follows:

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The goal of this phase is to determine the optimal mandibular advancement position. Each set of aligners is recommended to be worn at night for 10 days, for a total wear time of 8 hours per night. Please follow your doctor's specific instructions.

· Phase 2: Teeth Correction + Primary Snoring/OSAHS Treatment

The treatment goal of this phase is minor anterior alignment. The wear schedule for Clear Aligners and Sleep Aligners is synchronized, and each pair is suggested to be worn for 7-10 days.

Clear Aligners: Wear during the day for 10-12 hours.

Sleep Aligners: Wear at night for 8 hours.

· Phase 3: Primary Snoring/OSAHS Treatment

The objective of this phase is to improve nighttime snoring.

Each pair is recommended for 90 days, with a total wear time of 8 hours per night.

Tips

- When replacing aligners, switch to the next pair in sequential order.
- Do not switch to previous aligners unless instructed by your doctor.
- Follow your doctor's instructions for aligner changes. Each pair should not be worn for more than 10 days.

Use of Aligner Seaters

- After the aligner is in place, pinch one end of the seater and bite gently.
- Move the seater to various areas to ensure each tooth is engaged.
- Bite down 3–5 times to ensure the aligner fits your teeth perfectly.

Tips

- Please do not use the seater until the aligners have been correctly inserted, as premature biting may cause damage to the aligners.
- Rinse the seater with running water after use and store it in aligner case to prevent loss or contamination.

Daily Care and Maintenance

Cleaning

- Clean aligners with a toothbrush and toothpaste every day.
- You can also soak your aligners in a solution containing one Clear Aligner Cleansing Tablet for a few minutes before brushing them.
- Please do not clean your aligners in hot water which may cause deformation of your aligners.

Storage

- After cleaning, dry and store your aligners in the aligner case at room temperature (20–40°C).
- Keep aligners away from chemicals, pets, and young children, to prevent any potential damage.
- Store used aligners in a clean, resealable plastic bag for temporary use in case your current pair is lost or damaged.
- Do not wrap your aligners with a napkin or tissue. Many aligners have accidentally been thrown out this way.
- Clean and dry the aligner case regularly to prevent proliferation of bacteria.

Follow-up Visits

- Pay regular follow-up visits to your doctor, typically scheduled every 6–8 weeks.
- Consult your doctor for specific visit times and frequency.

FAQs

Q What are attachments?

A Attachments are small ridges made of dental materials which are attached to the teeth. They are customized and bonded on the tooth surface temporarily to guide the tooth movement and improve aligner retention. Attachments are bonded or to removed at different times according to the treatment needs, and will be removed after treatment.

Q What if an attachment falls off?

A If an attachment falls off during the treatment process, please contact your doctor for re-bonding.

Q What should I do with my previous aligners after each change?

A We recommend you keep your previous aligners in a safe place as a backup in case of accidental loss or damage of your current aligners, so that you can wear the previous aligners from the last stage before using a replacement.

Q What if my aligners are missing?

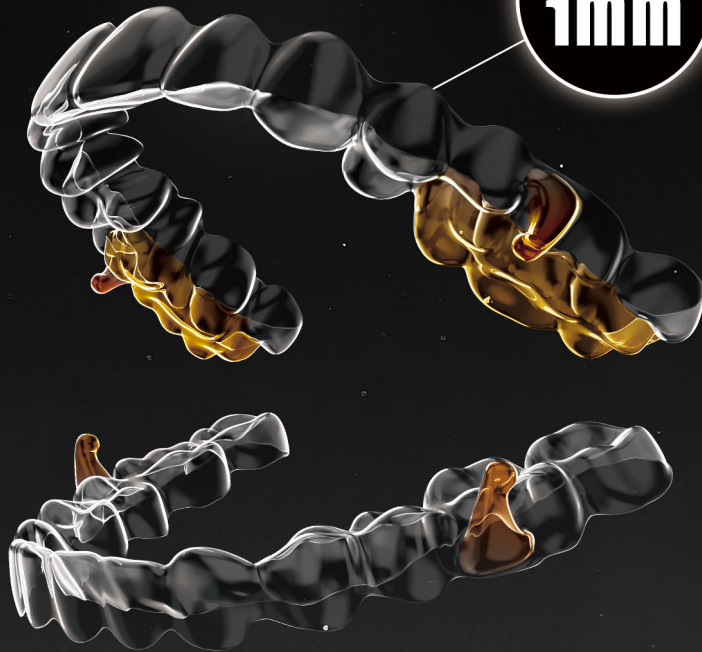
A You may wear the set of aligners for the last stage and consult your doctor for advice in the meantime.

Orthodontics & Anti-Snoring

Improving sleep health while
shaping an ideal smile

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